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W A L M A R K* Dispatch note of print out packing material W A L M A R K*

Design / Year: Chamot / 2015 DTP operator / Studio: (ext-THA) Jiri Sotek / Internal Proof numb. / Date 1. proof: 4/11. 10. 2022
Used printing material: BO 60 g Special printing technology: - Used Colours: 4 Cut Size (mm): 160 x 210 mm

C Y A N

M A G E N T A

Y E L L O W

B L A C K

RELIEVES AND PREVENTS THE SYMPTOMS OF ALLERGIES

PLEASE READ THIS INSTRUCTION FOR USE CAREFULLY

What is Sinulan Forte Allergy and what is it used for?

Sinulan Forte Allergy relieves the symptoms of allergies caused by pollen, dust mites or pets by forming a barrier against the penetration of airborne allergens. Sinulan Forte Allergy is a decongestant nasal spray indicated to improve the symptoms and course of chronic and acute sinusitis, rhinosinusitis, allergic rhinitis and nasal congestion (blocked nose, including colds) with a moisturizing barrier effect relieving the irritation for the restoration and maintenance of a healthy nasal mucosa.

It can also be taken as a preventive measure before coming into contact with an environment where airborne allergens are likely to be present (flowering periods, gardening activities, home dusting, etc.) from the very beginning, before symptoms appear.

How to use Sinulan Forte Allergy

Always administer Sinulan Forte Allergy according to the instructions for use. Consult your healthcare provider or pharmacist before taking it in case of doubt or if symptoms persist.

Apply 1-2 puffs in each nostril 3 or 4 times a day. Regular use is recommended when first symptoms are likely to appear and during the course of rhinitis. The dosage may be increased according to severity of symptoms.

1. Gently blow your nose to clean your nostrils.
2. Shake the bottle and remove the protecting cap.
3. Hold the bottle with your index and middle finger on both sides and your thumb underneath the bottle. Bend your head slightly forward and close one nostril by gently pressing against the side of your nose with the finger.
4. Insert the applicator into your nostril and press down on the nozzle to puff once or twice.
5. Insert the applicator into your other nostril and repeat the same steps described above.
6. Recap the bottle.

The nozzle used in Sinulan Forte Allergy provides a droplet size suitable to ensure a narrow jet thus allowing the adequate distribution of the solution in the whole nasal cavity, reaching even the upper part.

Possible side effects of Sinulan Forte Allergy

Sinulan Forte Allergy is very well tolerated. Up to now, no regularly occurring or persisting side effects are known. Please inform your healthcare provider if you notice side effects that are associated with the use of Sinulan Forte Allergy.

Precautions

Sinulan Forte Allergy should not be used in case of hypersensitivity to xyloglucan or to any other ingredient of the product.

If allergic reactions occur, the use of Sinulan Forte Allergy must be discontinued immediately.

To ensure safe hygiene, each bottle should only be used by one person.

Do not use for more than 6 weeks after first opening.

Do not use the device if damaged.

Sinulan Forte Allergy does not contain any pharmacological ingredient such as antihistamines or steroids. Should another pharmacological treatment be needed, Sinulan Forte Allergy can be administered concomitantly with it. Even though no side effects are known in children, consult your healthcare provider or pharmacist for advice before using this medical device if you are under 6 years old.

What should be noted during pregnancy and while breast-feeding?

Even though no side effects are known, consult your healthcare provider or pharmacist for advice before using this medical device if you are pregnant or breastfeeding.

How to store Sinulan Forte Allergy

The expiry date of Sinulan Forte Allergy is printed on the box and on the bottle.

Sinulan Forte Allergy must not be used after the expiry date.

Sinulan Forte Allergy does not require any special storage conditions. Do not freeze.

Sinulan Forte Allergy should be kept out of the reach of children.

Composition:

Ingredients: Xyloglucan, sodium benzoate, potassium sorbate, sodium EDTA, sodium hydroxide and saline solution.

Presentation:

Bottle containing 15 ml solution for nasal application.

Importer:

STADA (Thailand) Co., Ltd.
8 T-one Building, 18th Floor, Unit 5, Soi Sukhumvit 40,
Sukhumvit Road, Phra-Khanong, Klongtoei, Bangkok
10110 Thailand.
E-mail: stada@stadathailand.com
Tel: 0-2059-7420-23

Manufacturer:

DEVINTEC SAGL
Corso Elvezia 14, 6900 Lugano - Switzerland